

Device Description

Tasso+ is a sterile, disposable blood lancing device. When collecting a sample, the device is held onto the skin by a mild adhesive. A sterile lancet punctures the skin with a press of the button, simultaneously this creates a slight vacuum that helps capillary blood flow under gravity into a compatible tube. After a predefined time period, or when the tube is full, the device is peeled off the skin. The tube is removed and capped for transport. The lancet is single use and automatically retracts into the device in a safe position after use.

Materials Provided

- Tasso blood lancing device (FDA 510(k) - cleared)
- Instructions for use

Materials Required (Not Provided)

- Gloves (for healthcare providers)
- Compatible tube, refer to manufacturer’s labeling
- Heat pack or other arm warming method
- Bandage
- Alcohol swab

Intended Purpose

The Tasso+ is a single-use blood lancing device intended for obtaining microliter capillary whole blood samples.

Symbols

REF

Catalog number

LOT

Lot number

Use by date

Single use only

Store at 18-28°C

Transport at 25-85% RH

Consult instructions for use

This device contains sharps

Do not use if package is damaged

STERILE R

Sterilized by irradiation

Do not resterilize

Rx ONLY

Prescription use only

Disposal

Follow local guidelines for discarding used sharps as a biohazard material.

Model	Length	Depth
Tasso+	3.8 mm	0.8 mm

Warnings

Not intended for more than one use. Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested.

Precautions

- For use only on a single patient. Discard the entire device after use.
- Do not use after expiration date.
- Single use only.
- Keep out of reach of children.
- Not for use on infant heels.
- Prescription use only.
- Do not resterilize.
- For external use only.
- Do not use if device packaging has been opened or damaged.
- Use while seated as fainting may occur with any blood sampling procedure.
- Refer to manufacturer's instructions for individual components.
- Use personal protective equipment when handling biohazardous materials.
- Users must validate use of product for their specific assay.
- Minor bruising, residual marks, or scarring may occur at the sample collection site.

Note: The occurrence and severity of these events depend on patient physiological characteristics and chosen anatomic site.
- Multiple collections from the same anatomic location may increase the risk of residual marks or scarring and may impact the healing process.
- Do not perform collections on an area that shows evidence of skin issues such as infection, inflammation, extensive scarring, or broken skin.
- Patients taking blood thinners may experience prolonged bleeding.



First time using Tasso?

Watch our instructional video:
www.tassoinc.com/tassoplus-video

or scan the QR code to access.

Contact us for help:
1-800-257-2370

Additional Supplies

Razor
Shaving excess arm hair will improve the chances of a successful collection.

1 Prepare

Step 1

Wash hands before use. Healthcare workers must wear gloves during collection.

Step 2

Device will be placed on upper arm - for best results remove any excess hair.

Step 3

Collect all needed items. Remove device from tray.

Step 4

Twist off cap from compatible tube.

Step 5

Press tube into device until snug with fill lines facing out.

Step 6

Follow heat pack instructions to activate. If pack is not available, use other warming method such as hand.

Step 7

Increase blood flow at collection site
Rub heat pack or hand up and down shoulder for 2 minutes to warm and increase blood flow.

Step 8

Clean area with alcohol pad and allow to dry.

Step 9

Remove clear cover from red button.

2 Collect

Step 10

Peel tab away from device and stick device to upper arm. Hang arm straight down at side.

Step 11

Press button all the way down ONCE and release. Set timer for 5 minutes. Watch tube fill using mirror as needed.

Step 12

You may not feel the device. Leave on 5 minutes or less if blood at top fill line. Blood may not appear for first 1-2 minutes. Remove device at 5 total minutes or sooner if blood reaches top fill line before 5 minutes.

Step 13

Remove device by peeling from one side. Apply bandage vertically and add slight pressure.

Step 14

Remove tube from device with a slight twist and pull down.

Step 15

Snap cap fully onto tube.

Step 16

Invert tube per manufacturer's labeling.

3 Disposal & Transport

Step 17

Follow local guidelines for discarding all materials.

Step 18

Wash hands after use.

Step 19

If returning blood sample to a lab by parcel service, follow provided shipping instructions.

